Leva, chair mount Ergonomic arm support

17008

17005

17006

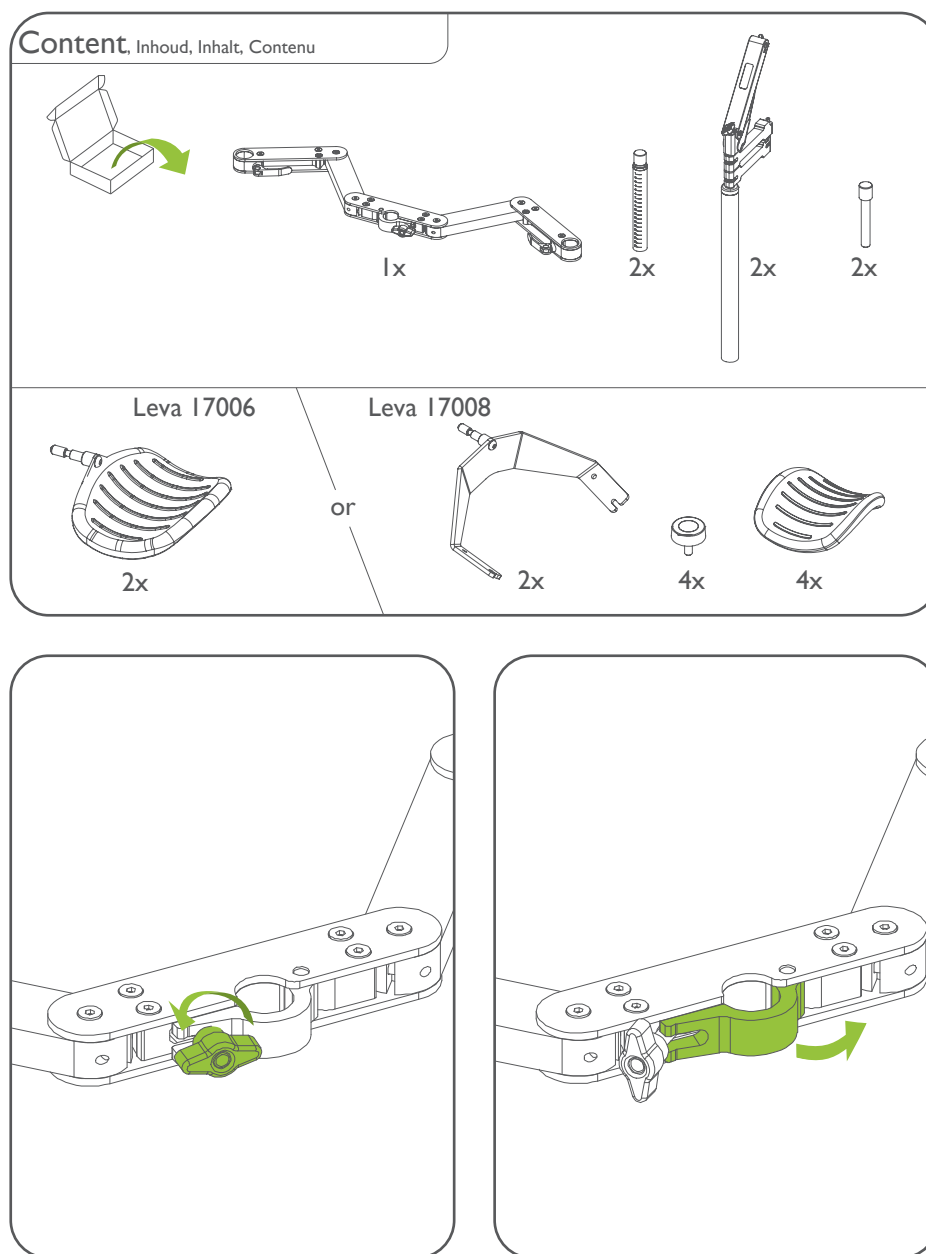
Installation manual

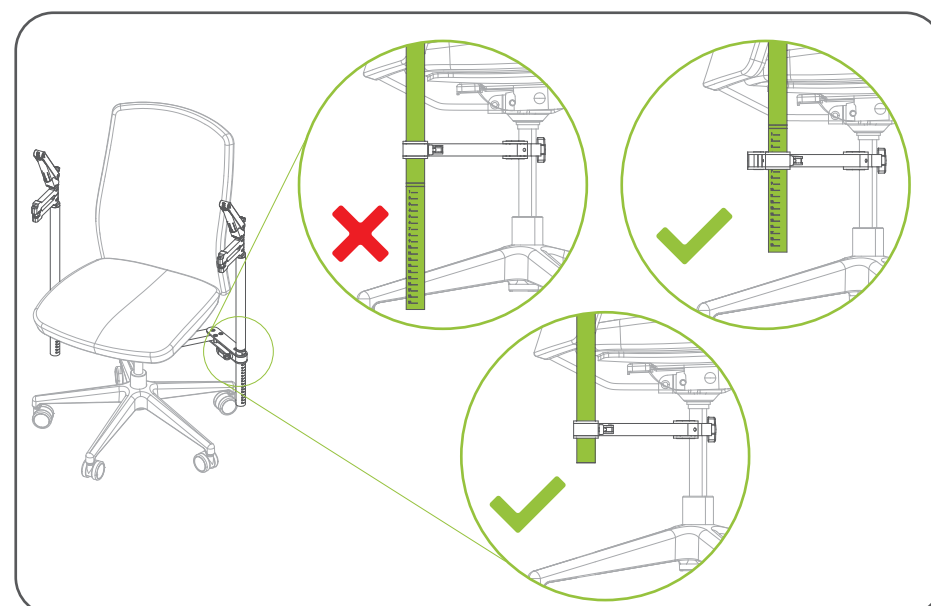
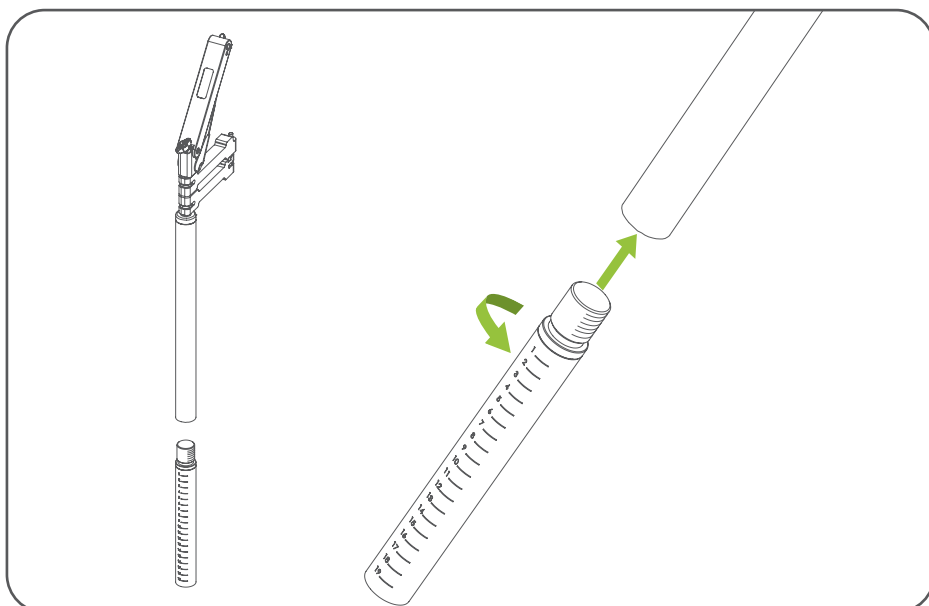
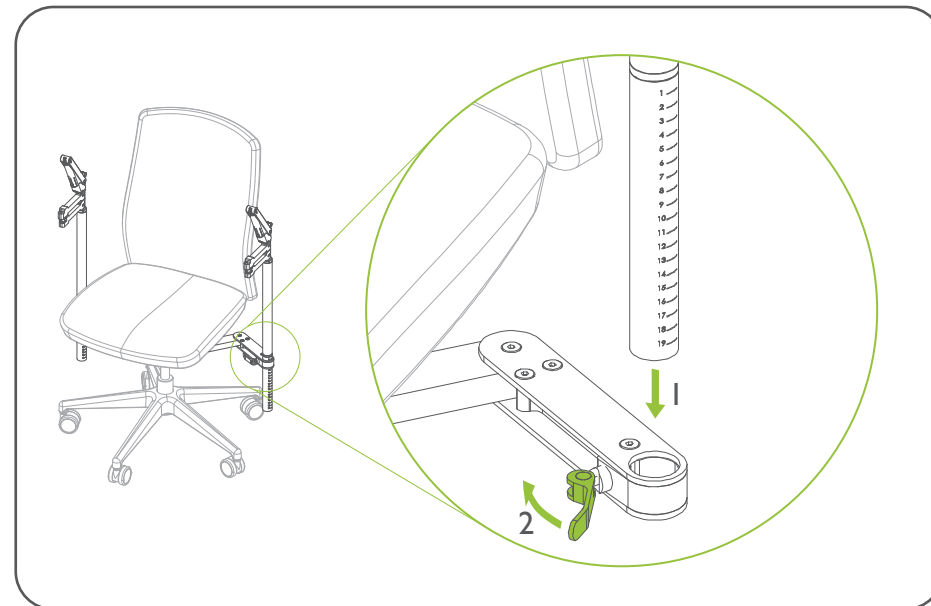
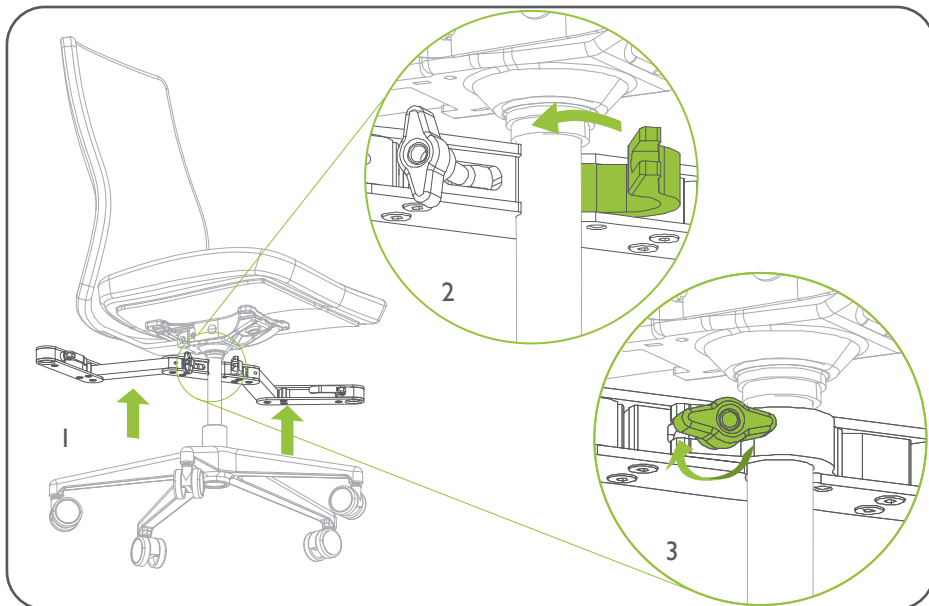
EN	Assembly instructions
NL	Montage-instructie
DE	Montageanleitung
FR	Instructions de montage

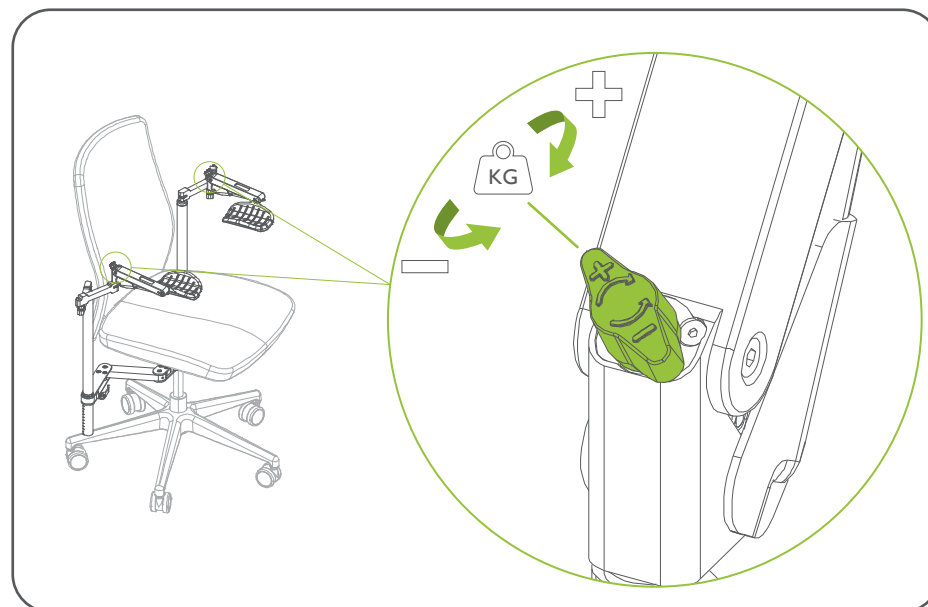
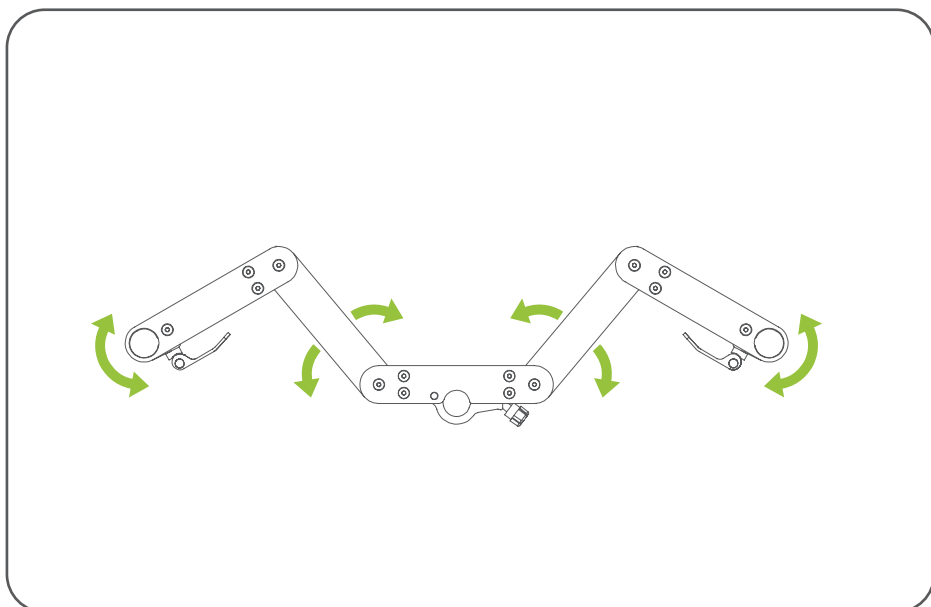
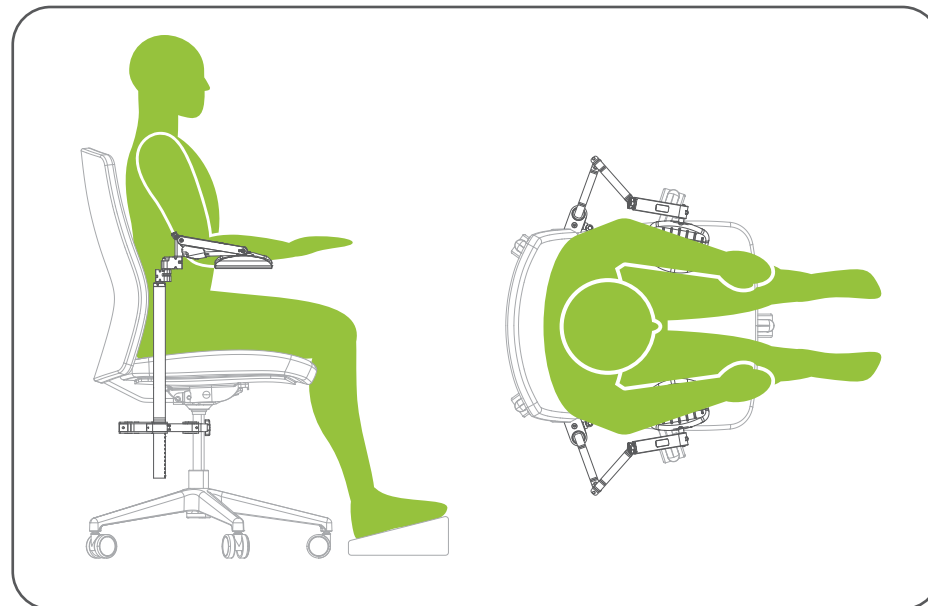
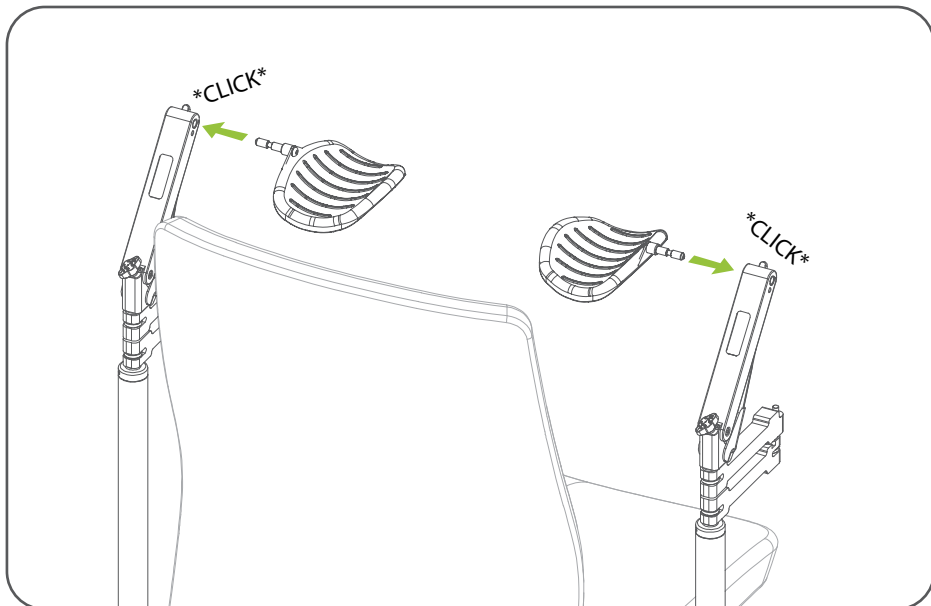
Contents	3
Leva I7007 / Leva I7008	6
Leva I7007	10
Leva I7008	14
Leva I7005 / Leva I7006	18
Leva I7005	21
Leva I7006	26
Important information (EN)	29
Belangrijke informatie (NL)	32
Wichtige Informationen (DE)	35
Information importante (FR)	



Leva I7007 / Leva I7008





Leva I7007

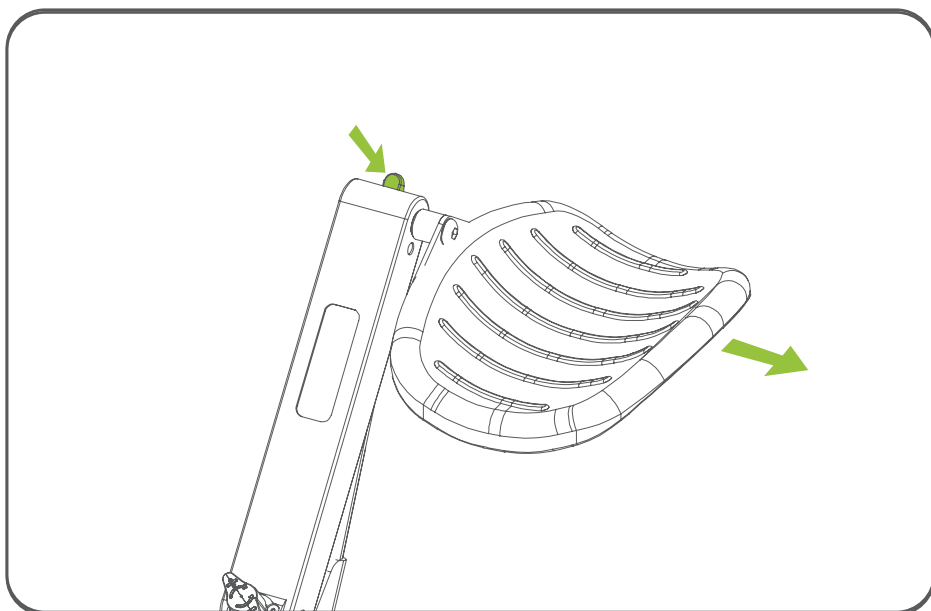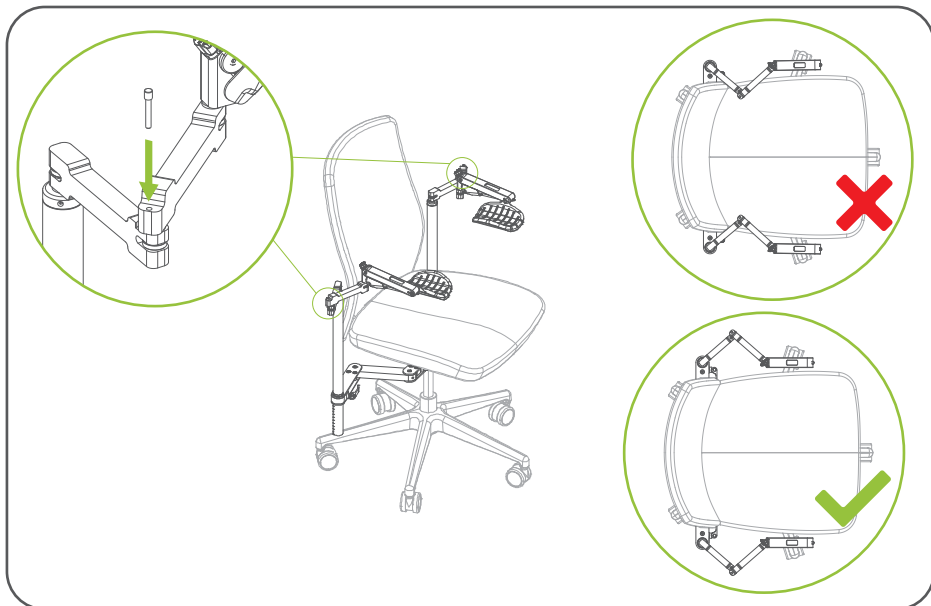Leva I7007


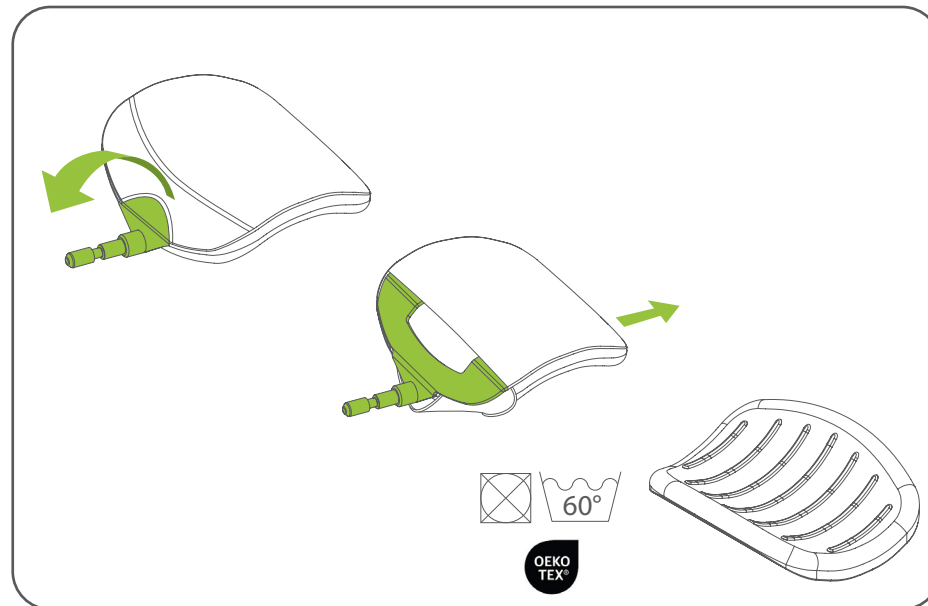
Leva I7008






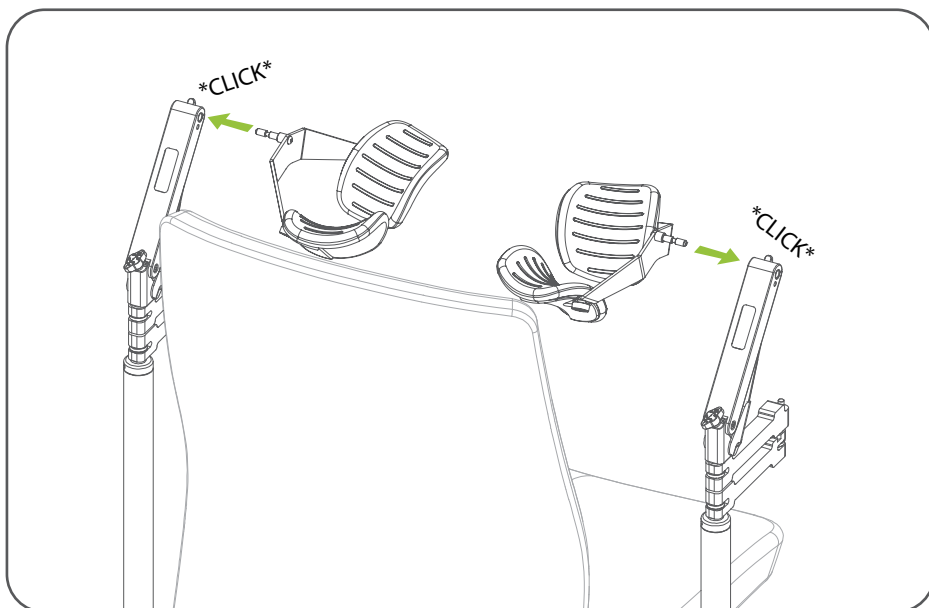
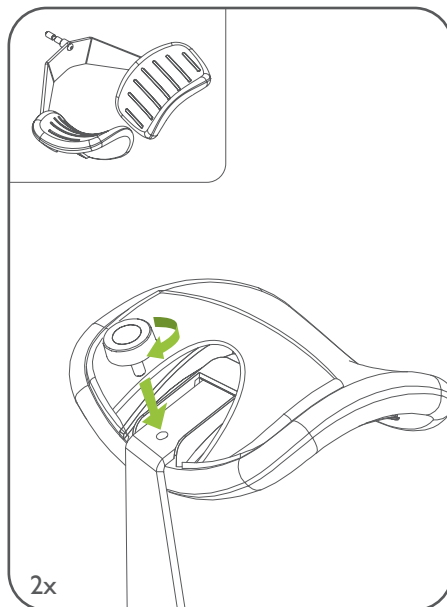
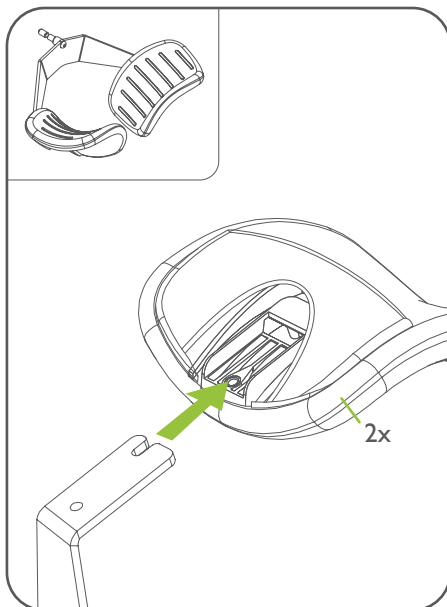
10



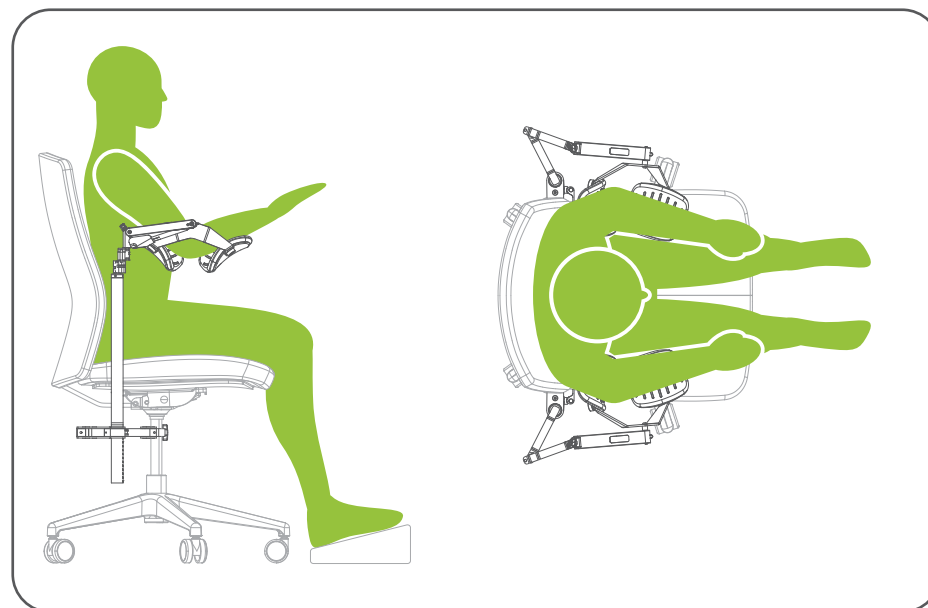
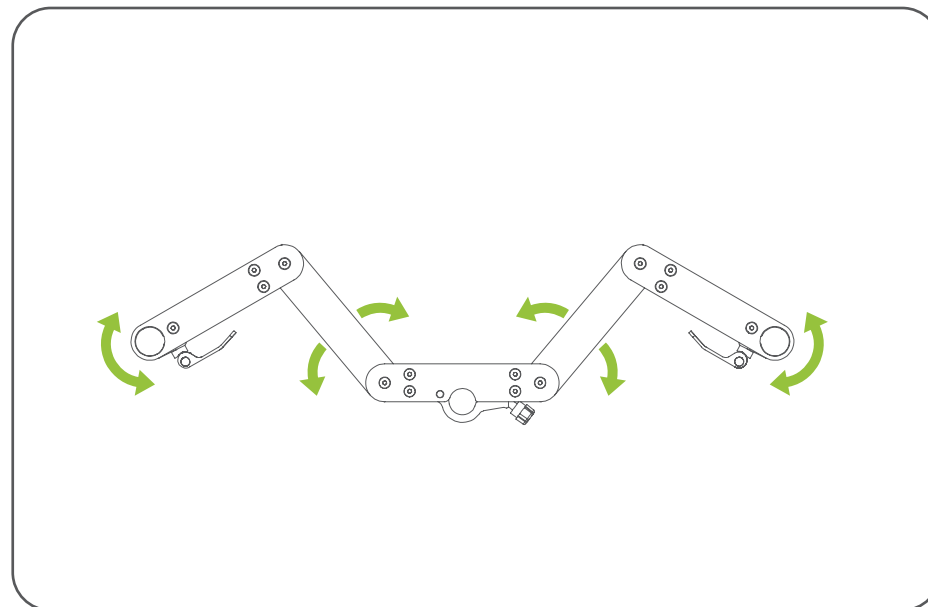
Leva I7008





11

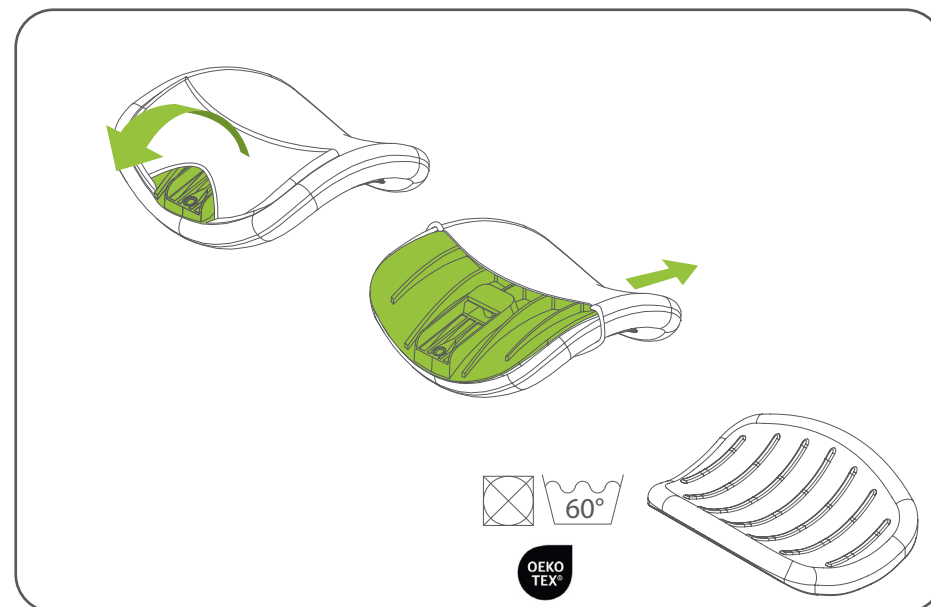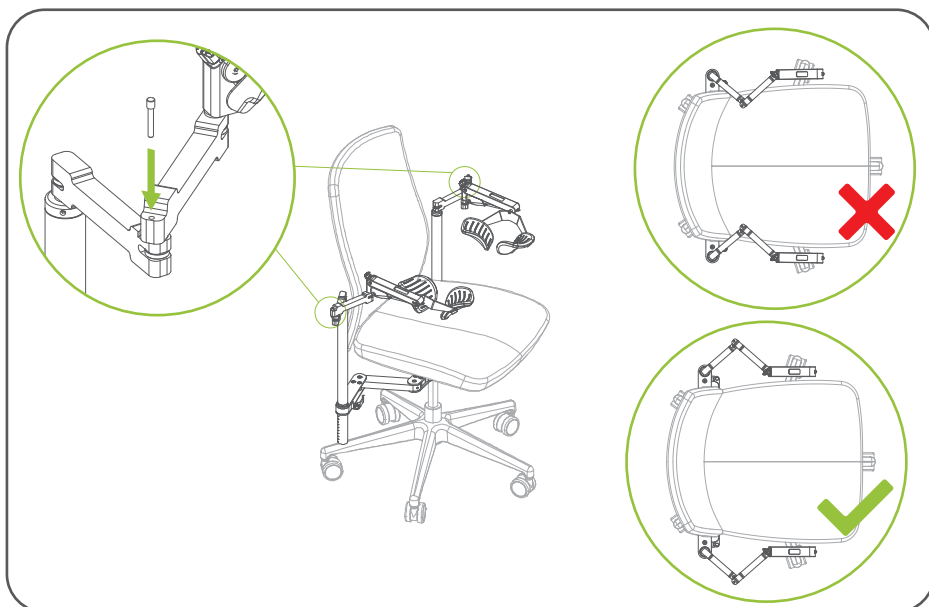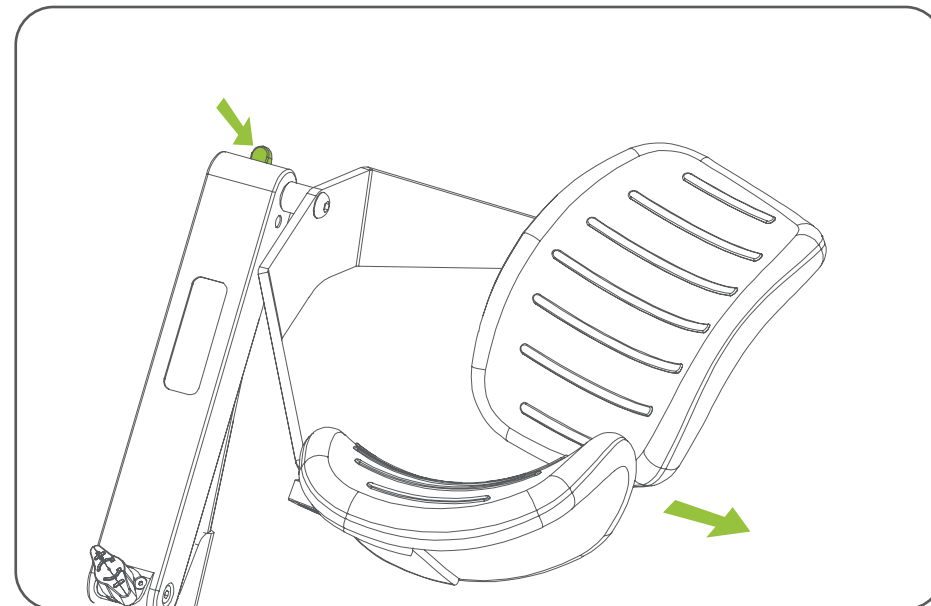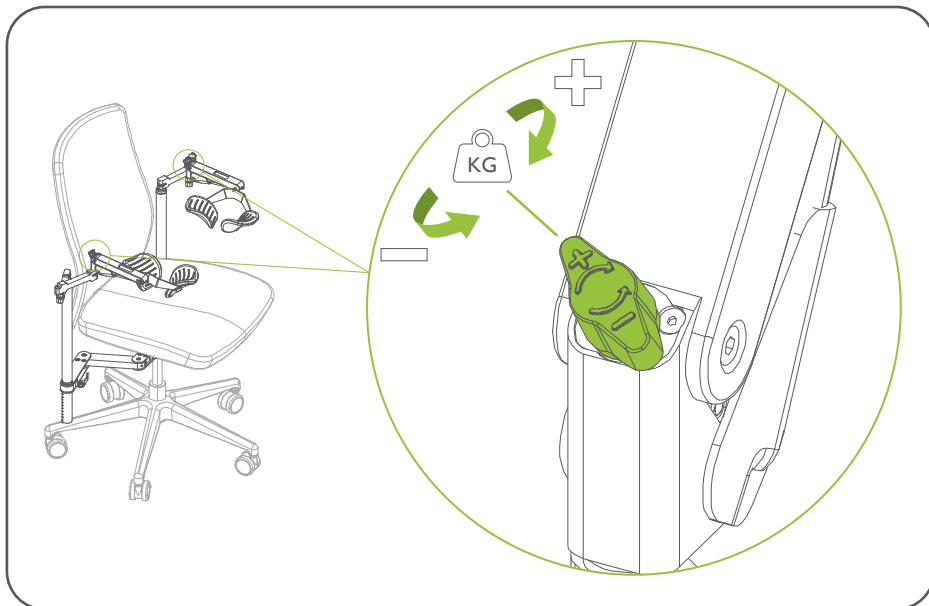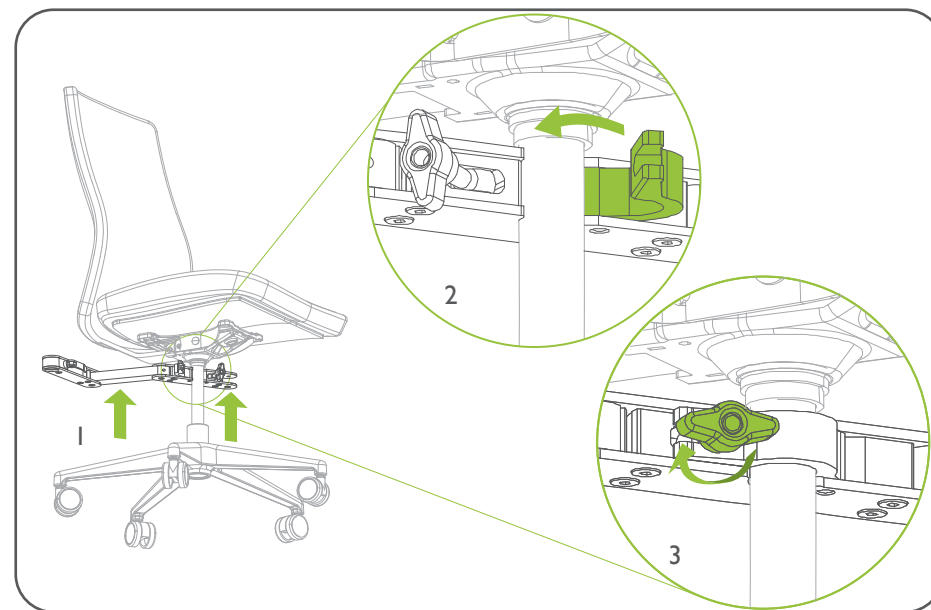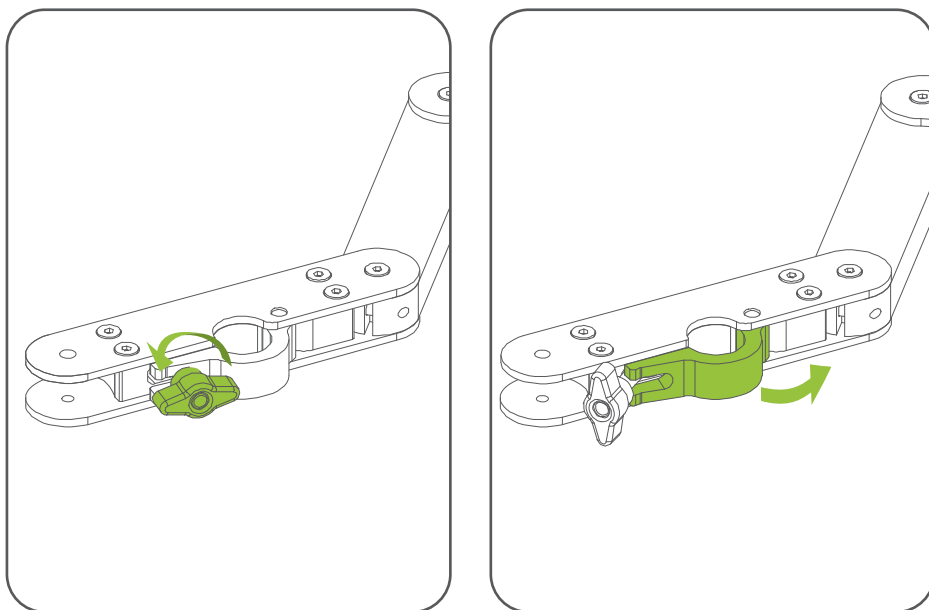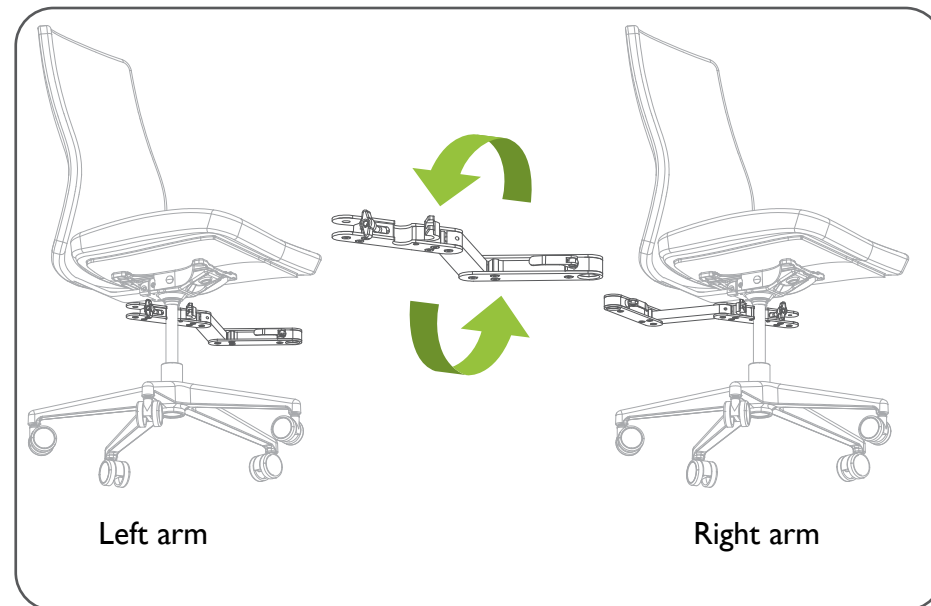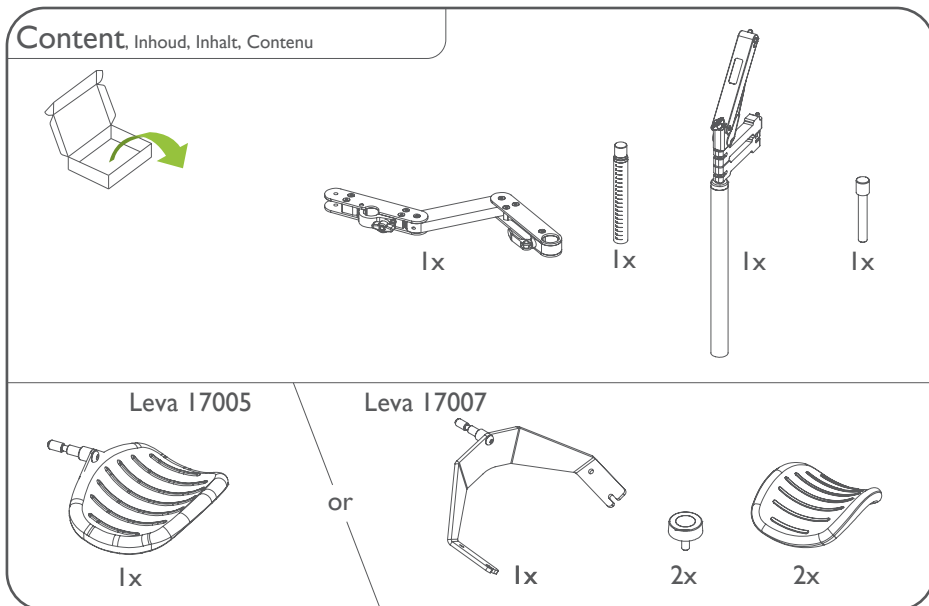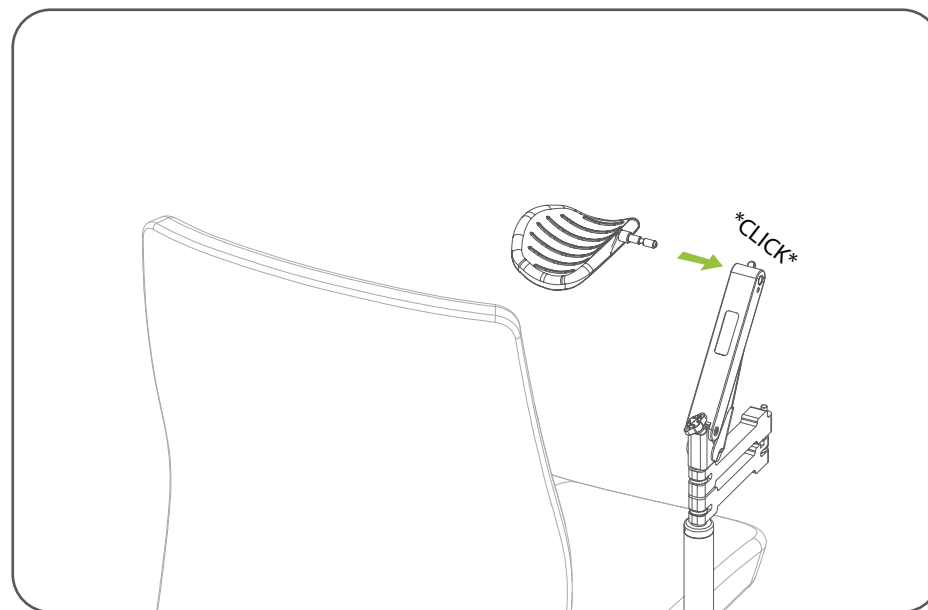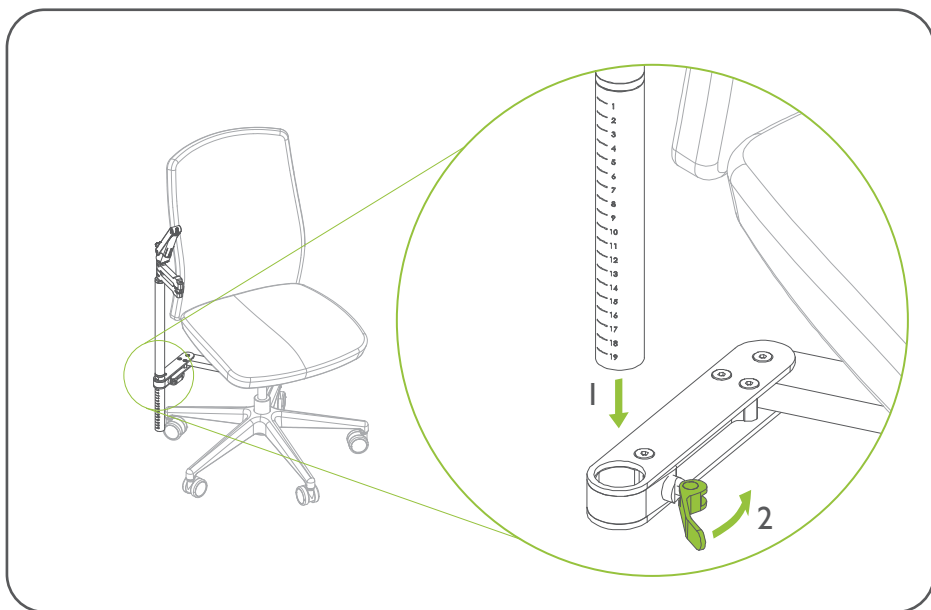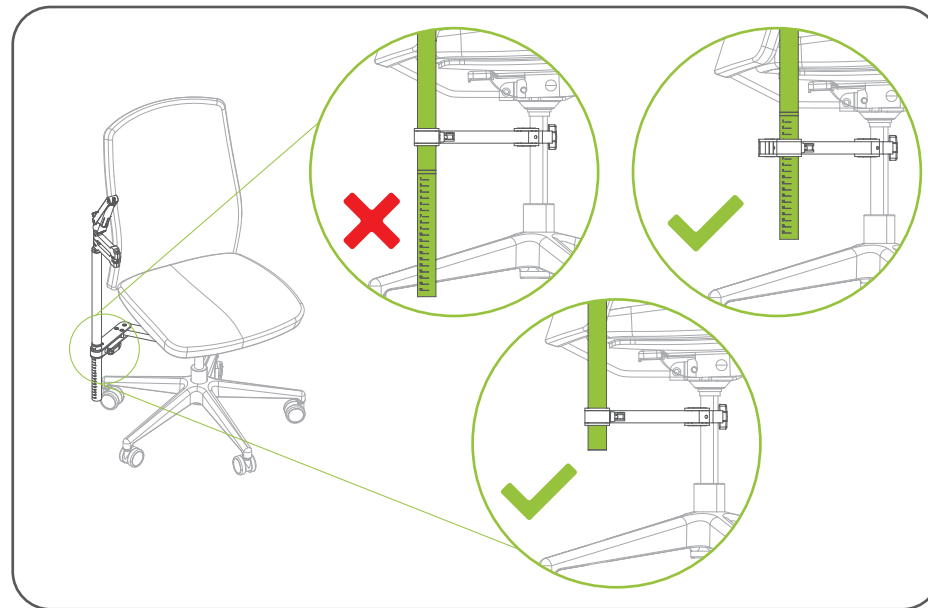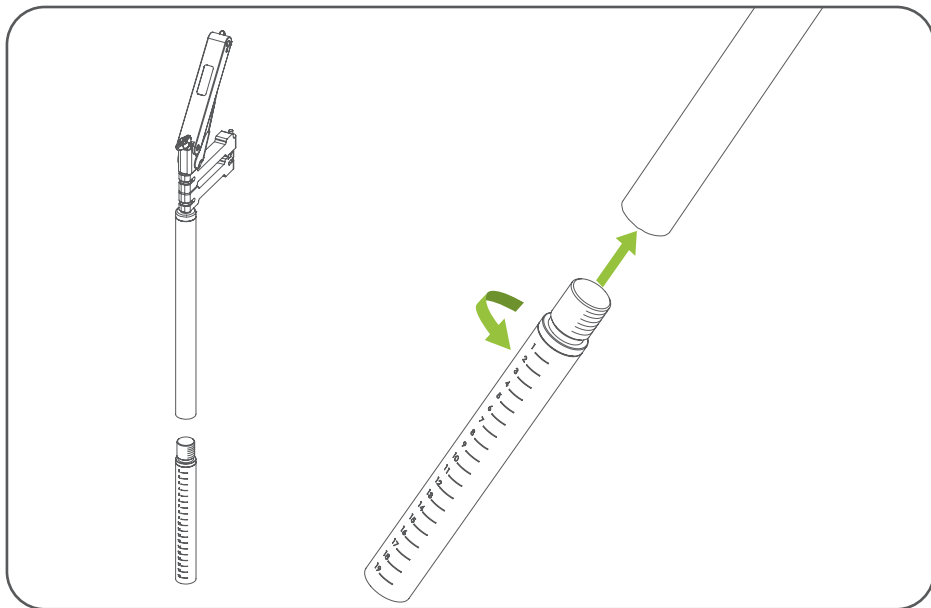
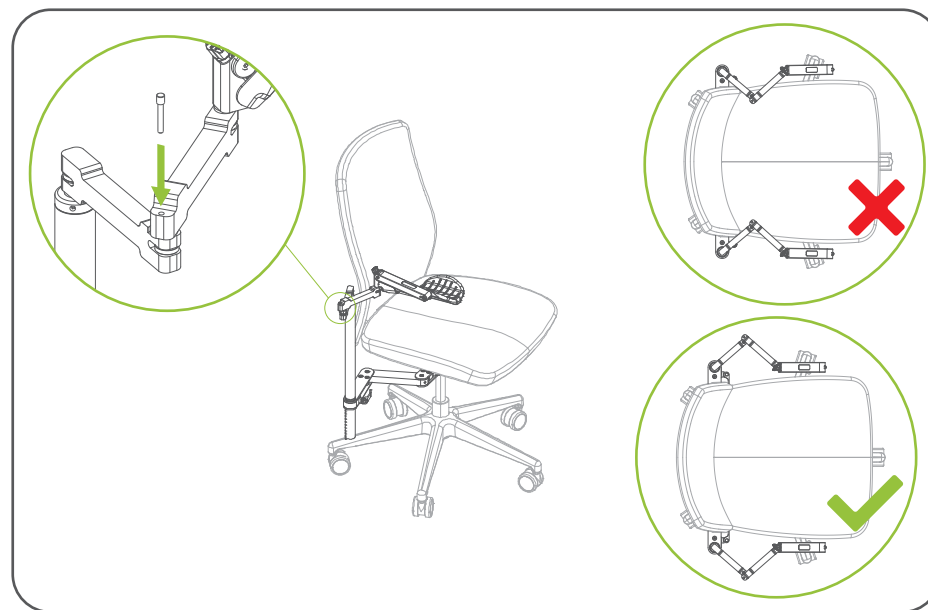
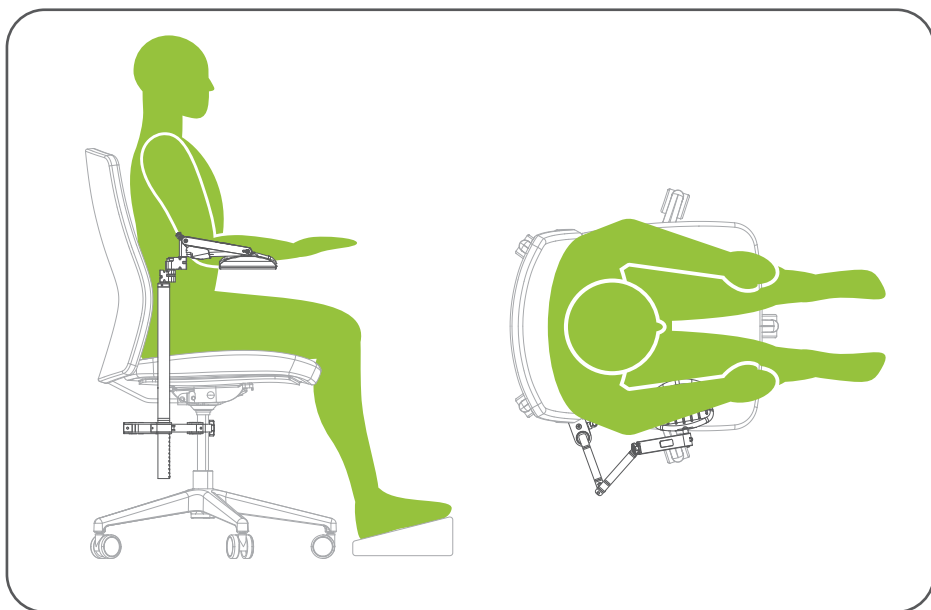
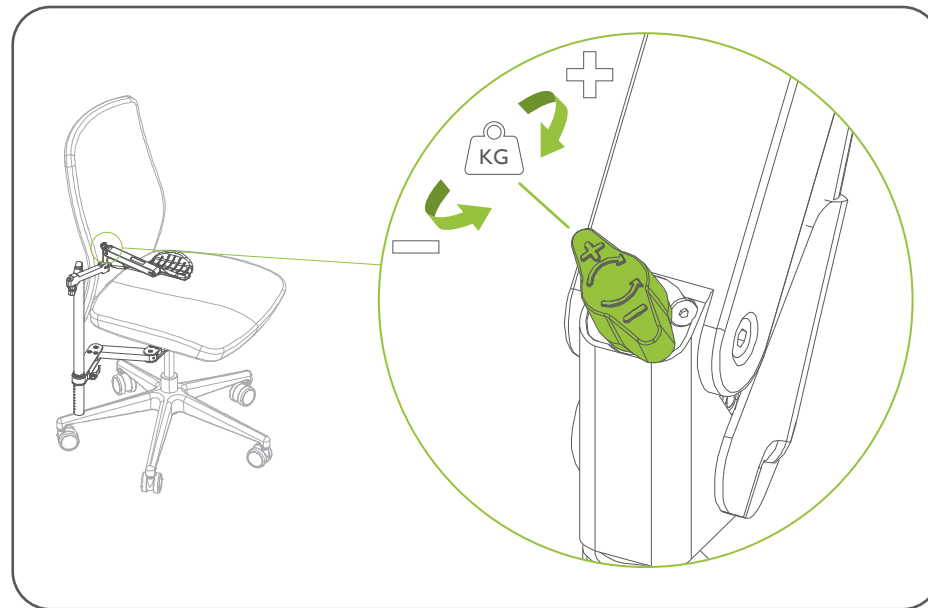
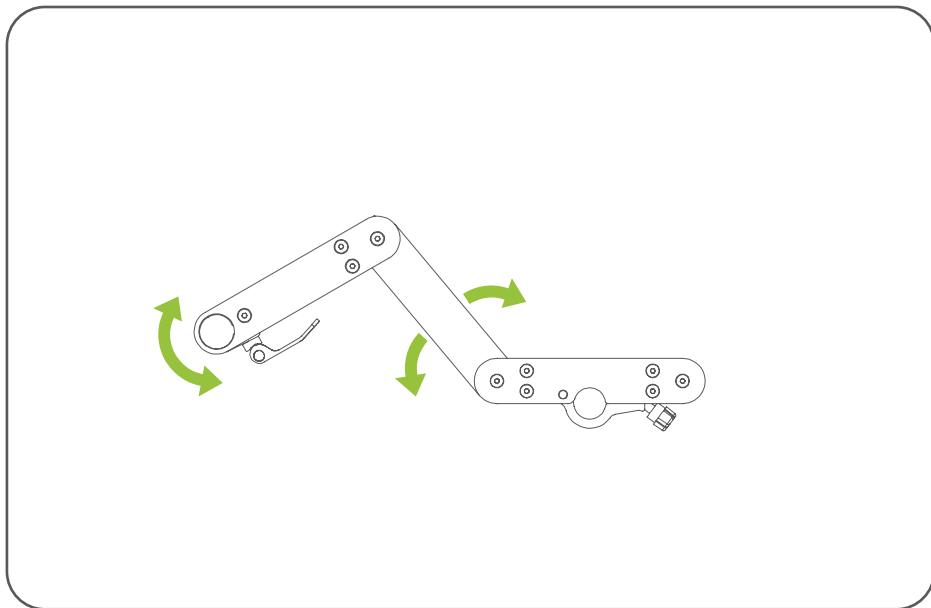
Leva I7005





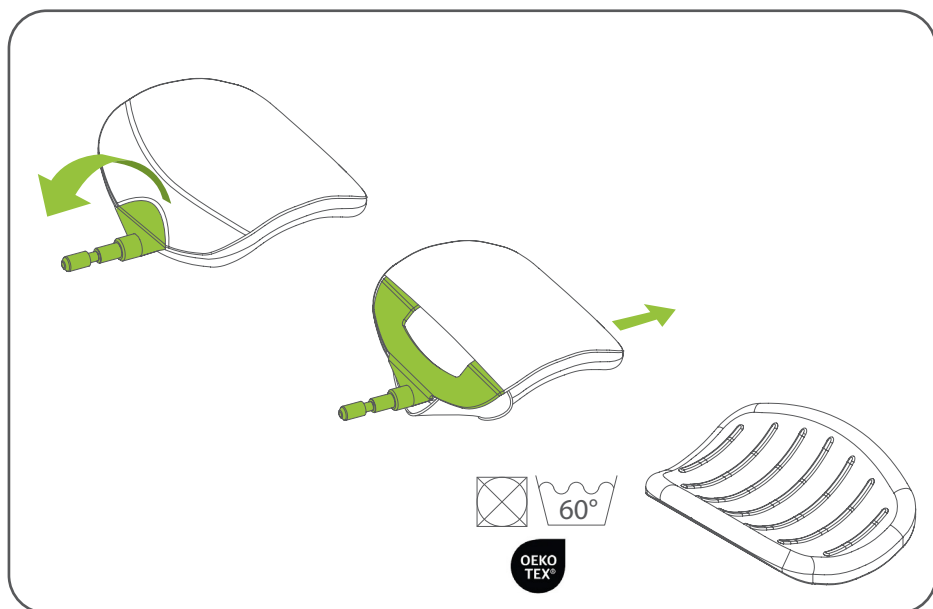
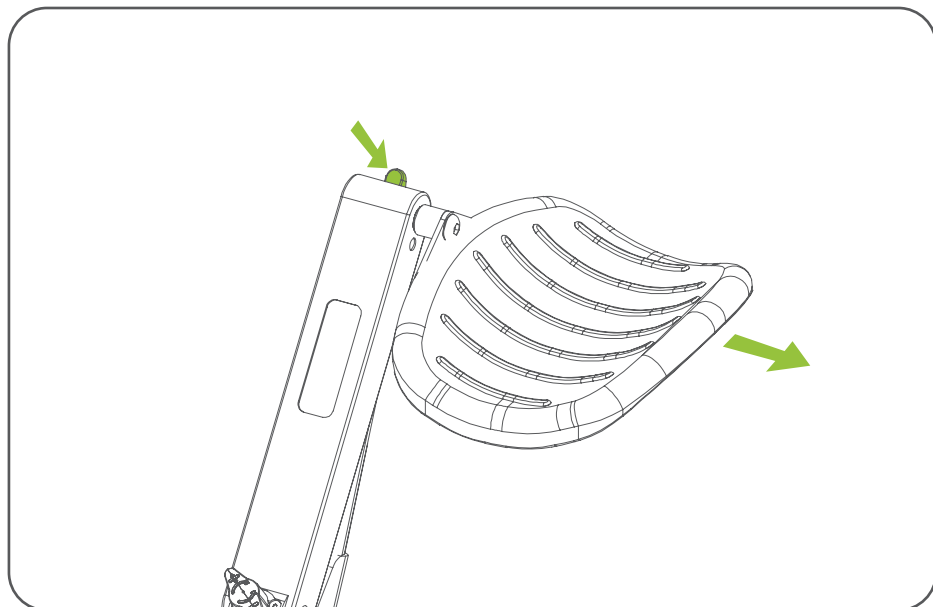
20

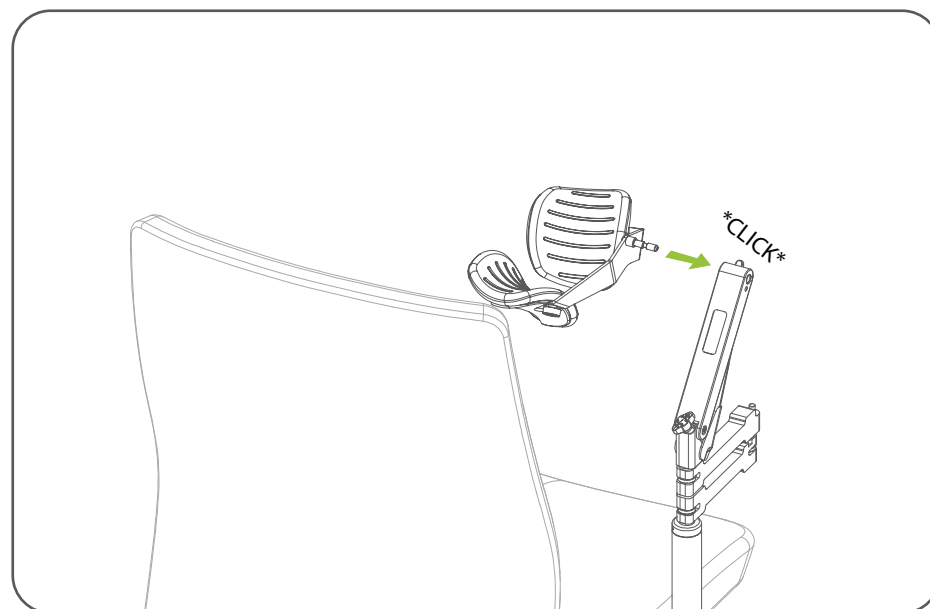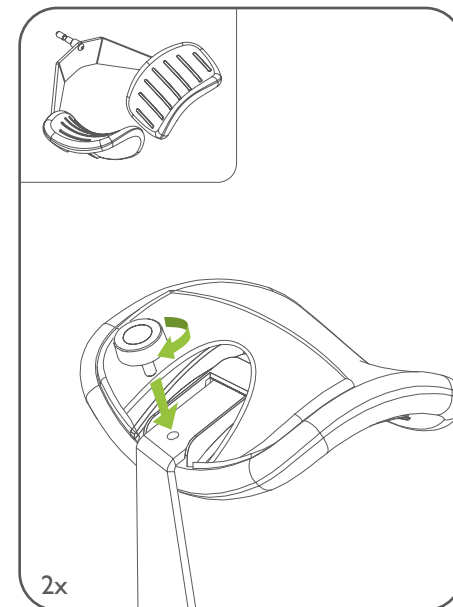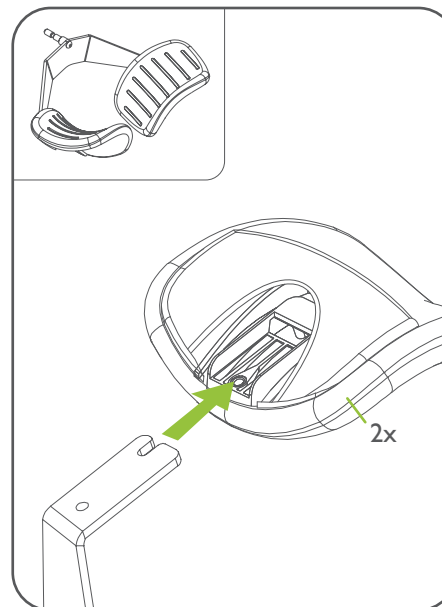Leva I7006

21

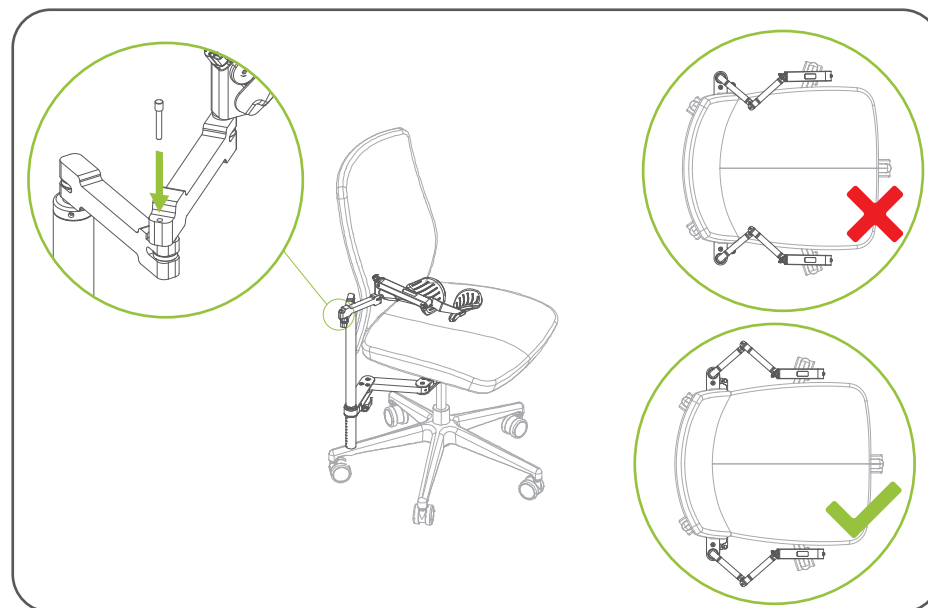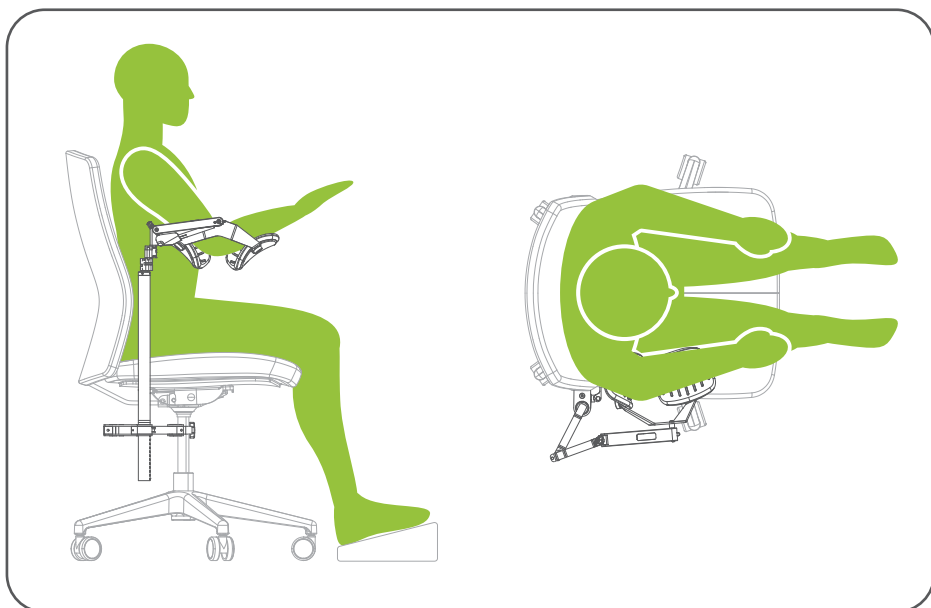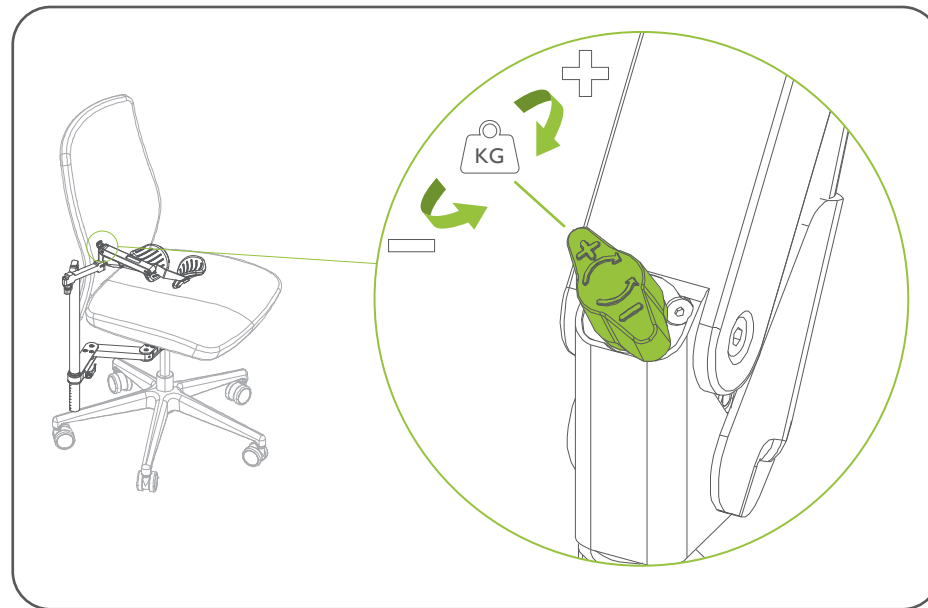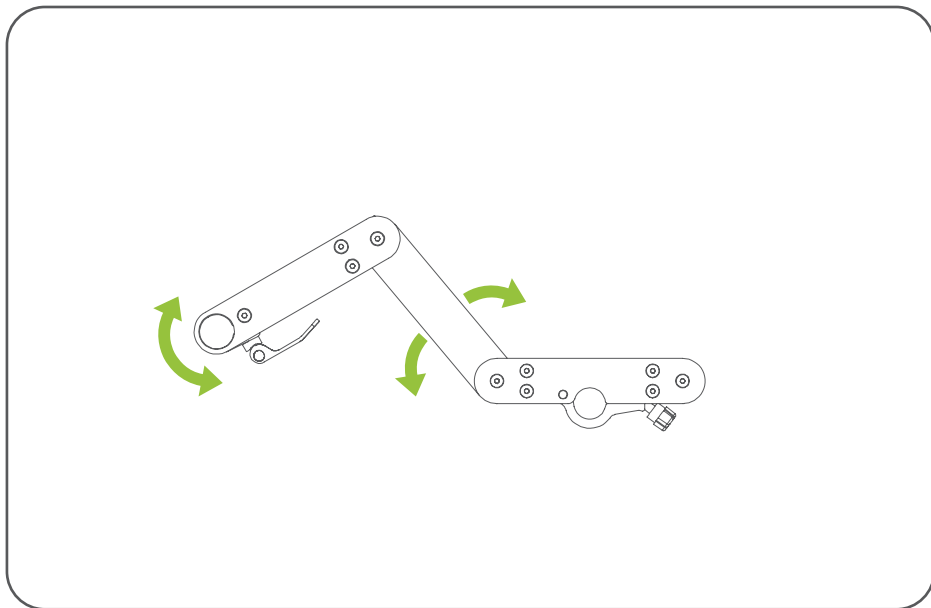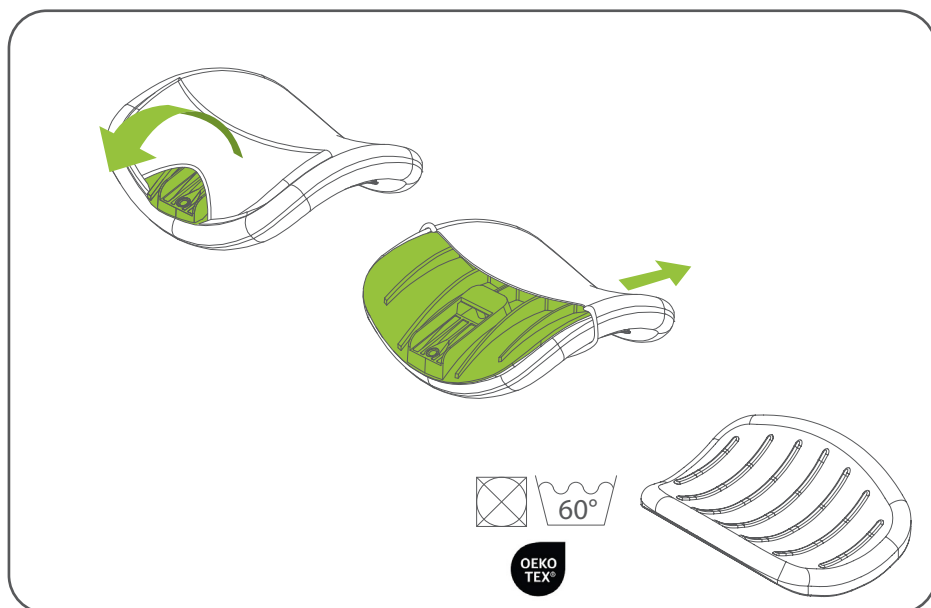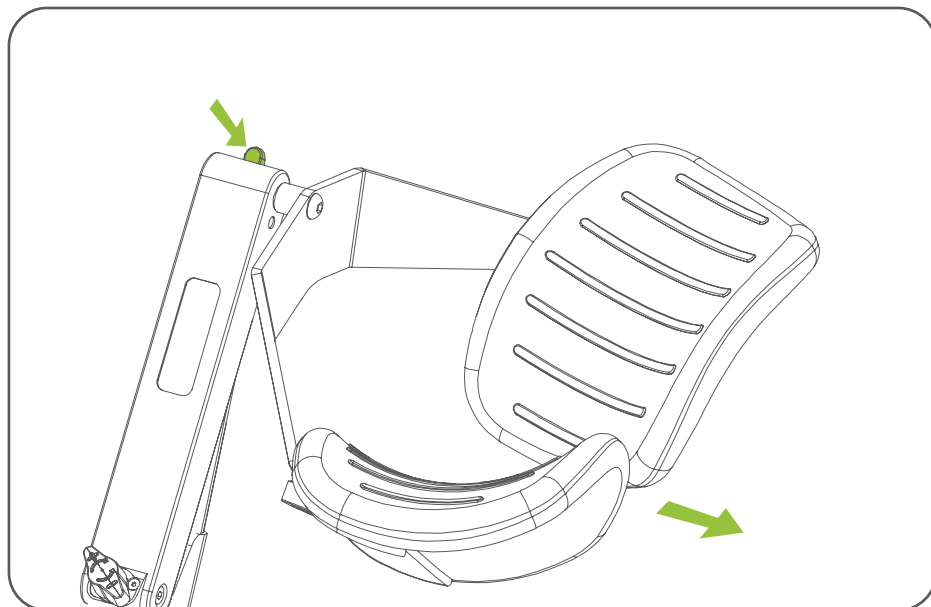Foreword

Congratulations on your purchase! This manual has been written for the owners and operators of the Leva dynamic arm support. It contains general instructions on operation, precautionary practices, maintenance and parts information.

Before using the product please make sure that you read the complete manual thoroughly. This is essential for your own safety and in order to maximize the use, functionality, efficiency and lifespan of your unit.

The content of this manual is for informational use only and is subject to change without notice. Armon Products BV assumes no responsibility or liability for any errors or inaccuracies that may appear in this document. Periodically, changes may be made to the information in this manual; these changes will be incorporated into new editions of this publication. Changes to the information in this manual or the products may be made at any time without obligation on the part of Armon Products BV. The complete user manual can be downloaded on our website. In case you want to receive a (printed) user manual in your own language, please contact us.

Symbols used:

 warning / caution / important

 specialized disposal guidelines

Intended use:

The Leva is a dynamic arm support which reduces physical complaints caused by repetitive movements or a static posture during work. It should only be used to support an arm with a maximum weight or downwards force of 6000 grams. This weight equals the maximum compensation setting of the product. Do not exceed this maximum weight. It cannot be used as a support when standing up and sitting down.

Product lifespan and maintenance

The Leva is designed to guarantee a products lifespan of at least 6 years under normal conditions. The Leva is a maintenance free product within the expected technical lifespan. However, wear and tear of the moving internal mechanics depends on the intensity of use and the environmental circumstances. Protect it against sand, vapours, and dust. Careless use or disregarding normal clean handling principles can lead to repairs or a shortened product lifespan.

Cleaning and disinfection

When cleaning the outer parts of the Leva, except for the armpad cover, make sure to use a soft, clean cloth dampened with lukewarm water and a non-abrasive liquid household cleaner.

To disinfect the outer parts of the Leva, except for the armpad cover, use a cloth (non-abrasive) with a standard available disinfectant based on 70% alcohol / ethanol / isopropylalcohol.

Solvents can be used but they can affect the appearance and lifespan of the surface treatment / coating. The armpad cover can be washed up to 60°C or 140°F, and is OEKO-TEX certified.

Warranty and conditions

In the case of direct supply by Armon Products BV to end users, warranty and conditions and subject to the Consumer General Terms and Conditions issued by Koninklijke Metaal Unie and in accordance with Dutch law. In case of supply through a representative of Armon Products BV, warranty and conditions are subject to conditions of the national or local representative and in accordance with national law.

Warranty claims for these products shall be voided in case: the conditions in the operating manual are ignored; any part is modified or opened; improper use; the product has been set up incorrectly; any use outside its intended use; the product label has been removed.

Liability

To the maximum extent permitted by applicable law, in no event will Armon Products or its suppliers or resellers be liable for any indirect, special, incidental or consequential damages arising from the use of or inability to use the product, including, without limitation, damages for loss of goodwill, work and productivity, computer failure or malfunction, or any and all other commercial damages or losses, even if advised of the possibility thereof, and regardless of the legal or

equitable theory (contract, tort or otherwise) upon which the claim is based. In any case, Armon Products entire liability under any provision of this agreement shall not exceed in the aggregate the sum of the fees paid for this product and fees for support of the product received by Armon Products under a separate support agreement (if any), with the exception of death or personal injury caused by the negligence of Armon Products to the extent applicable law prohibits the limitation of damages in such cases. Armon Products cannot be held liable for any consequence resulting from incorrect information provided by its personnel, or errors incorporated in this manual and / or other accompanying documentation (including commercial documentation). The opposing party (product's user or its representative) shall disclaim Armon Products from all claims arising from third parties, whatever nature or whatever relationship to the opposing party. Any severe incident in relation to this product should be mentioned to the manufacturer and the competent national authority.

Safety

The Leva stores mechanical energy for balancing the arm. During non-intended removal of the arm from the arm pad or arm brace, this mechanical energy will be released. Resulting in a fast upwards accelerating movement which can result in bodily harm. When removing the arm from the support, do so slowly after

bringing it to its most upright position. Never use it as a catapult.

Be aware of finger pinching. Never stick your fingers in any cavity or between (movable) parts of the product. Make sure people around you, particularly children, do not put their fingers near pinch points.

The arm support may only be used for the intended use. Any type of use which is not specified in this manual will be considered as improper use. The customer is solely responsible for material damage and injury of persons resulting from improper use.

Should you experience any problems with the installation, or any discomfort whilst using your device, please contact Armon Products immediately.


End of life

The Leva contains materials that can be recycled and/or are noxious to the environment. Specialized companies can dismantle the unit and sort out these materials. When you dispose of the unit, find out about local regulations concerning waste management

Product specifications

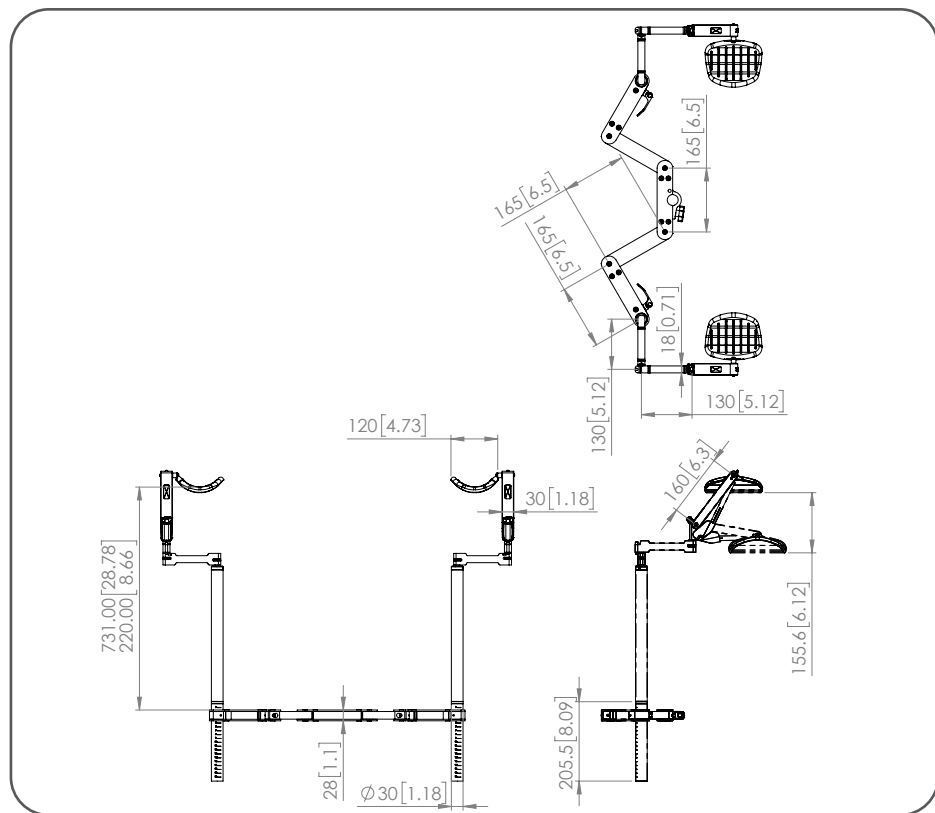Arm weight compensation range	1,5 kg to 6 kg / 3.3 lbs to 13.2 lbs
Maximum payload	6 kg / 13.2 lbs
Weight	Leva 17005/17006: 3,4 kg / 7.5 lbs Leva 17007/17008: 5,8 kg / 12.8 lbs
Device structure material	aluminium
Operating / storage temperature range	-10°C to 40°C / 14°F to 104°F
CE marking	Medical Device class I
Device in conformity with	MDR EU 2017/745
Serial Registration Number Eudamed	NL-MF-000002064
UDI-DI	08720634841573
Declaration of Conformity	available upon request

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Armon Products B.V.